

# **IMPORTANT NOTICE**

## **Possible effect of power failures to the High/Low temperature recording function of ICS PACIFIC Medical Fridges.**

### **Models:**

ICS PACIFIC G 135 Medical

ICS PACIFIC G 375 Medical

### **Problem -**

Following restoration of power after a power failure there can be a 2 hour delay before the high/low temperature recording function resumes normal operation.

During this 2 hour delay the high/low temperatures are not recorded so it is possible that the reading will display the same high/low temperature reading as before the power failure.

### **Correct operation -**

For the safe keeping of medicines the fridge needs to record the correct high/low temperature readings immediately after restoration of power.

### **Solution -**

The following simple instructions need to be carried out once only - not after every power failure.

### **Please note:**

If no button is pressed for 10 seconds the display will return to normal. If this happens restart the procedure from 1 below.

**1.** Press and hold the Set button for approximately 6 seconds - the display will change to E1.

**2.** When E1 is displayed press and release the Set button until C3 is displayed.  
(the Set button needs to be pressed 10 times to reach C3)

**3.** When C3 is displayed press and release the up button once (top left position). The display should be '00'.

**4a.** If the display is '00'

DO NOT DO ANYTHING as the fridge is set correctly. No other action is required.

In approximately 10 seconds the display will automatically return to normal.

**4b.** If the display is not '00'

Press either the up or down button (top or bottom left positions) as required until the display reads '00'.

DO NOT DO ANYTHING else as the fridge is set correctly. No other action is required.

In approximately 10 seconds the display will automatically return to normal.

The fridge will now record the correct high/low temperature readings immediately after restoration of power.